



VD^x Cleaner - P

Intended Use: Cleaner - P is used for clean the bath & tubing in Prokan 3- part differential hematology analyzers.

Pack Size: 1 x 10 L

Active Ingredients:

- o Sodium Chloride: 0.5%
- o Sodium Sulphate: 1.2%
- o Triton X 705: 0.8%
- o 2-phenoxy ethanol 0.8%
- o Proxol-100 0.5%
- o Tris Buffer: 1.0%, pH 7.5
- o Preservative: 0.05 %

Directions for Use:

Open the container cover. Pull the connector of the container out and connect it with the corresponding Diluent tubing as per instructions given in the Analyzer's user manual.

Precautions:

1. For in-vitro diagnostics use only.
2. Do not inhale the reagent or pipette by mouth.
3. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
4. Do not use if package is damaged.
5. Dispose waste product in accordance with local Govt. regulations.

Storage and Stability:

The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C - 30°C. Use Cleaner - P within 6 months of first opening.

MLC.No. : MFG/VD/2021/000002

LOT :

EXP :

M.R.P. (Rs.) :

(Incl. of all Taxes)

Manufacturer : Vanguard Diagnostics Private Limited

C-122, Phase I, Okhla Industrial Area
New Delhi-110020, INDIA.
Tel: 011-4172 7222. Email: info@vanguardiagnostics.com



PM303008L 148