



Cleaner - S

Intended Use : Cleaner – S is used for periodic cleaning of the tubes & bath of Sysmex 3-part differential hematology analyzers.

Pack Size: 1 x 100 ML

Active Ingredients:

Sodium Hypochlorite Solution : 0.4%(free chlorine 5%)
Sodium Hydroxide : 0.2%

Directions for Use:

Open the container cover of Cleaner – S during the system shutdown, proceed as per instructions given in the user manual of the Analyzer.

Precautions:

1. For *in-vitro* diagnostics use only.
2. Do not inhale the reagent or pipette by mouth.
3. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
4. Do not use if package is damaged.
5. Dispose waste product in accordance with local Govt. regulations.
6. Cap the bottle tightly after use.

Storage and Stability:

The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C - 30°C. Use Cleaner - S, within 6 months of first opening.

M.L.C.No. : MFG/IVD/2021/000002

[LOT] :

[MFG] :

[EXP] :

M.R.P. (Rs.) :

(Incl. of all Taxes)



Manufactured by:
Vanguard Diagnostics Private Limited
C-123, Phase-1, Okhla Industrial Area
New Delhi-110 020, INDIA.
Tel:011-4172 7222, Email: info@vanguarddiagnostics.com



PM30008L20