



# VD<sub>X</sub><sup>TM</sup>

# Diluent - A

**Intended Use:** Diluent - A is used for Diluting the blood in Horiba-ABX 3-part differential hematology analyzers.

**Active Ingredients:**

- Sodium Chloride: 0.8%
- Sodium sulphate: 1.2%
- EDTA Disodium Salt: 1.0%
- Triton X-705: 0.5%
- N.N. Dimethyl Urea: 0.6%
- Sodium Azide: 0.05%

**Directions for Use:**

Open the container cover. Pull the connector of the container out and connect it with the corresponding Diluent tubing as per instructions given in the Analyzer's user manual.

**Precautions:**

1. For *in-vitro* diagnostics use only.
2. Do not inhale the reagent or pipette by mouth.
3. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
4. Do not use if package is damaged.
5. Dispose waste product in accordance with local Govt. regulations.

**Pack Size: 1 x 20 L**

**Storage and Stability:**

The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C - 30°C. Use Diluent - A, within 6 months of first opening.

**MLC.No.** : S(0079)/15/MB

**LOT** :

**M** :

**D** :

**M.R.P (Rs.)** :  
(Incl. of all Taxes)

**M** :

**Manufactured by:**  
Vanguard Diagnostics Private Limited  
C-123, Phase-1, Delta Industrial Area  
New Delhi-110 020, INDIA.  
Tel: 011-4172 7222, Email: info@vanguarddiagnostics.com



PM30008L156