



EZ Cleaner - M

Intended Use: EZ Cleaner - M is used for cleaning the tubes & bath before shutdown in Mindray 3-part differential hematology analyzers.

Pack Size: 1 x 100 ML

Active Ingredients:

- Sodium Chloride : 0.69%
- Sodium Citrate di-hydrate : 1.50%
- Calcium Chloride di-hydrate : 0.80%
- Enzymes : 0.02%
- Dye : 0.01%

Directions for Use:

Open the container cover of EZ Cleaner - M during the system shutdown, press the aspirate key to aspirate EZ Cleaner - M, as per instructions given in the user manual of the Analyzer.

Precautions:

1. For *in-vitro* diagnostics use only.
2. Do not inhale the reagent or pipette by mouth.
3. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
4. Do not use if package is damaged.
5. Dispose waste product in accordance with local Govt. regulations.
6. Cap the bottle tightly after use.

Storage and Stability:

The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C - 30°C. Use EZ Cleaner - M within 6 months of first opening.

MLC.No. : MFG/IVD/2021/000002

LOT :

EXP :

REF :

M.R.P. (Rs.) :

(Incl. of all taxes)

Manufactured by:

Vanguard Diagnostics Private Limited

C-123, Phase-I, Okhla Industrial Area

New Delhi-110 020, INDIA.

Tel: 011-4372 7222. Email: info@vanguarddiagnostics.com



PM30008L09