



VD_x™ Lyse - P

Intended Use: Lyse - P is used for lysing of cells for measurement of WBC & Hemoglobin in Prokan 3-part differential hematology analyzers.

Pack Size: 1 x 500 ML

Active Ingredients:

- o Quaternary Ammonium salt: 4.0%
- o Sodium Chloride: 1.0%
- o Imidazole: 1.2%
- o EDTA Disodium salt: 0.5%
- o Preservative: 0.01 %

Directions for Use:

Open the container cover. Pull the connector of the container out and connect it with the corresponding Lyse tubing as per instructions given in the Analyzer's user manual.

Precautions:

1. For *in-vitro* diagnostics use only.
2. Do not inhale the reagent or pipette by mouth.
3. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
4. Do not use if package is damaged.
5. Dispose waste product in accordance with local Govt. regulations.

Storage and Stability:

The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C - 30°C. Use Lyse - P within 6 months of first opening.

MLC.No. : MFG/IVD/2021/000002



:



:



:

M.R.P. (Rs.) :
(Incl. of all Taxes)



Vanguard Diagnostics Private Limited
C-122, Phase-1, DDA Industrial Area
New Delhi-110 020, INDIA.
Tel: 011-4172 7222. Email: info@vanguarddiagnostics.com



PK3000BL149